

Application MediZip

Preparation

- For deep incisions apply subcutaneous sutures as required before applying the MediZip.
- MediZip length must be a minimum of 4 cm longer than the wound.
- The Area surrounding the wound must be dry and cleaned of grease and exudate.
- Remove the MediZip from the sterile packaging.
- Separate the two sides of the MediZip by sliding the rider to the open position. Then remove the paper from one side.
- Place the adhesive side 2 mm from the edge of the wound and press on.
- Remove the protective film from the other side and apply also 2 mm from the edge of the wound

Closure

- The wound is closed by pushing the MediZip slider (photo). In order not to hinder the view of the wound pls. push the slider.
- Hold the MediZip at the end with one finger and slowly push up the MediZip slider until the MediZip is closed
- Apply protective dressing over the MediZip if necessary



- Leave the MediZip in place for 7-10 days
- Remove the MediZip in closed position.

Explanatory note:

Should an additional wound dressing be required, the slider portion of the device can be removed by cutting off the end stop, thus avoiding discomfort to the patient due to a feeling of pressure.

For external applications only – internal applications are prohibited.